


What is the difference between iQCath and iQCath+?

iQCATH®

Solo version requires water to activate coating (Running UK drinking Tap water is perfectly acceptable).

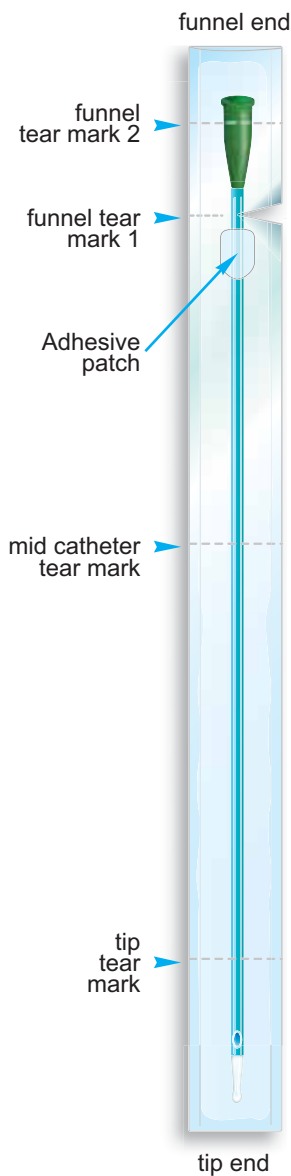

© Manfred Sauer UK Ltd.

✓ ADVANTAGE


They are easier to carry about your person.

✗ DISADVANTAGE

You need a source of clean water to activate the catheters coating.

iQCATH+®

Activating saline fluid included in packaging. Simply squeeze or roll-up foil sachet until it bursts and use fluid to activate catheter coating.

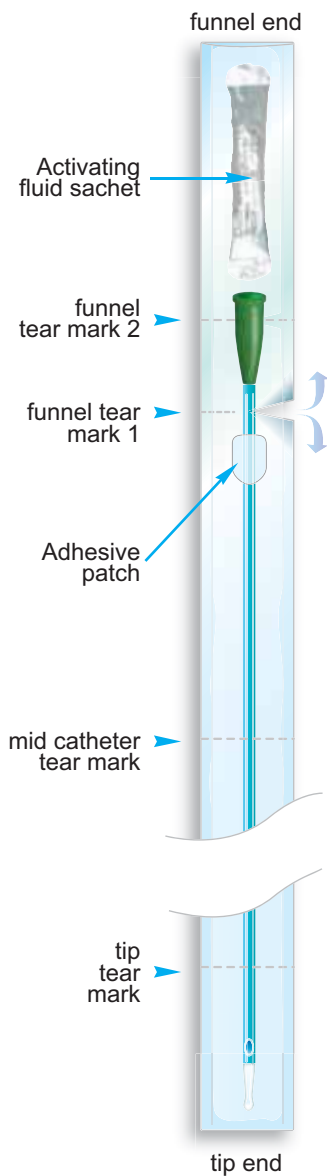

iQCath and iQCath+ prescription order information and codes on back page

✓ ADVANTAGE

No worries about finding clean water to activate coating.

✗ DISADVANTAGE

They are heavier and bulkier to carry than the solo version



Always store your catheters flat in a cool dry place away from direct sunlight. Catheters may be rolled not folded for carrying on the day they are to be used. Unroll any unused catheters at the end of the day.