


FITTING INSTRUCTIONS

p.sure

Latex-free

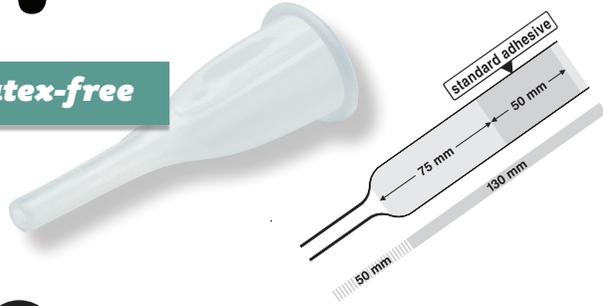

1 Correct Sizing

TO MEASURE: put tape around the middle of the penis so it is a snug fit.

Trim any pubic hair on the shaft and base of the penis. Don't shave as this can cause skin irritation. Use mild soap with no added moisturiser when washing this area & ensure dry before proceeding.

Tip: Use Kitchen roll to absorb moisture in the skins creases.

2 The Un-roll

2a) Rolled up sheath.

2b) Prior to fitting, the sheath must be unrolled at least 3 times.

2c) Then place on the penis. This leaves a "buffer /comfort zone" helping urine flow. The adhesive area starts further down the sheath. Use 2 smaller fingers to gently hold connector tube against the palm to create tension whilst rolling sheath on.

OPTIONAL - Use the protective cloth (free in each box of sheaths) to keep pubic hair away from the adhesive area.

3 The Buffer Zone

Unroll the rest of the sheath down the shaft of the penis, keeping it even and straight all the way down. **DO NOT** roll at an angle

WRONG as sheath unevenly rolled

4 The Sticky Bit

After the Buffer-Zone there's the adhesive area which should start behind the head of the penis.

The Buffer-zone
Adhesive area

5 Ensure Good Adhesion

Smooth out any wrinkles in the sheath. Hold the adhesive area of the sheath firmly for 15-30 secs. The warmth of your hand ensures the adhesive bond is strong.

6 The Connection

a) Fit sheath to the penis

b) Fit leg bag to leg comfortably & secure

c) Ensure the inlet tube is the correct length and not kinked - adjust length if required. Ensure sheath not twisted. **Only then connect inlet tube to sheath.**

Connection made easy if conical connector on leg bag is wet (optional)

DON'T TWIST

7 Sheath Removal

Remove by rolling off*

*Removal may be easier in bath or shower
Optional: Ideally remove after 24hrs to give your skin a chance to breathe

CE

MANFRED SAUER UK

Unit 3, io Centre, Lodge Farm,
Northampton NN5 7UW
Helpline Tel: 01604 595 696
Fax: 01604 588 091
email: helpline@manfredsauer.co.uk

www.manfredsauer.co.uk